

INNOLATEX (THAILAND) LIMITED (EU MDR)

NON-STERILE, WATER-BASED LUBRICANT

INSTRUCTION FOR USE

LEAFLET

T-IFU-02/1

Rev 12th January 2026 (v 13)

myONE™ Water-based Lubricant

DOES NOT CONTAIN SPERMICIDE. NOT A CONTRACEPTIVE.

myONE™ Lubricant is non-sterile, non-irritating, greaseless and water-soluble medical device to be used by lay persons – adult male or female. It is used as additional lubricant during intercourse for alleviation of vaginal dryness. It helps to reduce friction associated with thin and/or dry genital tissue, during sexual intercourse.

Direction/ Instruction for use: Squeeze/pump required amount of lubricant, typically about 2 to 5g per application, onto your clean fingers and/or apply directly onto vagina, penis or condom surface.

Storage: Store in a cool, dry place, away from direct sunlight

Ingredients: *Ingredient list shall be written according to lubricant variant formulation*

Warnings

- Avoid contact with eyes and ears.
- High glycol content may affect sperm motility. Avoid using this product if you are planning to conceive
- This product is not for ingestion. Keep out of reach of children. In the event of accidental ingestion, seek medical assistance immediately.
- Do not use during childbirth unless under medical supervision
- Do not use on infected, broken, breached, or injured vaginal or vulvar tissue. Consult your doctor before using this product.
- Do not use this product with other topical or systemic gynaecological medication. Product interactions are unknown, consult your healthcare provider before use.

Precautions



- This product is not a contraceptive and does not protect against pregnancy.
- This product does not contain microbicides and does not protect against sexual transmitted infection and HIV.
- Compatible with natural rubber latex and polyisoprene condoms.
- Not compatible with polyurethane condoms.
- Apply a small amount of lubricant on your inner thigh or forearm 24 hours before use. Check the area for any signs of irritation.

- Use with caution during menstruation and menopause, discontinue if irritation and discomfort occurs.
- If symptoms persist, immediately consult your doctor
- Effects of lubricant on pregnant and/or nursing woman are unknown. Consult your doctor before use.
- Use within 6 months after opening (**for lubricants in tube or bottle packaging only**)
- Do not re-use (**for lubricants in sachet packaging only**)
- User may reapply as needed to maintain lubrication during intercourse.
- Excessive use of lubricant may cause discharge and discomfort.
- Store in ambient temperature, dry place, away from direct sunlight
- Dispose empty packaging hygienically in a closed rubbish bin
- Report any serious incident that occurred from using the device to the manufacturer (www.karex.com.my/contact-us) and the relevant national competent authority

Contraindications

- There is no known effect of this lubricant on pregnant and/or nursing woman, woman with vaginal trauma/surgery and/or vaginal infection. Consult doctor before using this product.
- Not recommended for use in case of known allergies to any of the ingredients

Definition of Symbols (All symbols used on product label conform to the harmonized standards, except below with their description)

	Indicates that the product shall be used within 6 months after opening
	Indicates instruction to dispose empty packaging into rubbish bin

Innolatex (Thailand) Limited
E1-6, Export Processing Zone,
Southern Industrial Estate,
Village 4, Chalung Sub-district,
Hatyai District,
Songkhla, 90110,
Thailand

CE
0123

INNOLATEX (THAILAND) LIMITED (EU MDR)

NON-STERILE, SILICONE OIL-BASED LUBRICANT

INSTRUCTION FOR USE
LEAFLET

T-IFU-02/2

Rev 20th May 2026 (v5.0)



Silicone Oil-based Lubricant

DOES NOT CONTAIN SPERMICIDE. NOT A CONTRACEPTIVE.

myONE[®] silicone oil lubricant is non-sterile and non-irritating medical device to be used by lay persons - adult male or female. It is used as additional lubricant during intercourse for alleviation of vaginal dryness. It helps to reduce friction associated with thin and/or dry genital tissue, during sexual intercourse.

Direction/ Instruction for use: Squeeze/pump required amount of lubricant, typically about 2 to 5g per application, onto your clean fingers and/or apply directly onto vagina, penis or condom surface.

Storage: Store in a cool, dry place, away from direct sunlight

Ingredients: *Ingredient list shall be written according to lubricant variant formulation*

Warnings

- Avoid contact with eyes and ears.
- Silicone oil may affect sperm motility. Avoid using this product if you are planning to conceive.
- This product is not for ingestion. Keep out of reach of children. In the event of accidental ingestion, seek medical assistance immediately.
- Do not use during childbirth unless under medical supervision
- Do not use on infected, broken, breached, or injured vaginal or vulvar tissue. Consult your doctor before using this product.
- Do not use this product with other topical or systemic gynaecological medication. Product interactions are unknown, consult your healthcare provider before use.
- Silicone oil lubricant is extremely slippery on all surfaces with washed with soap and water
- Do not use if package is torn or broken

Precautions

- This product is not a contraceptive and does not protect against pregnancy.
- This product does not contain microbicides and does not protect against sexual transmitted infection and HIV.
- Compatible with natural rubber latex, polyisoprene condoms, and polyurethane male condoms

- Use with caution during menstruation and menopause, discontinue if irritation and discomfort occur, discontinue use immediately.
- If symptoms persist, immediately consult your doctor
- Effects of lubricant on pregnant and/or nursing woman are unknown. Consult your doctor before use.
- Use within 24 months after opening (**for lubricants in tube or bottle packaging only**)
- Do not re-use (**for lubricants in sachet packaging only**)
- User may reapply as needed to maintain lubrication during intercourse.
- Product may stain some fabrics
- Excessive use of lubricant may cause discharge and discomfort.
- Store in a cool, dry place, away from direct sunlight
- Dispose empty packaging hygienically in a closed rubbish bin
- Report any serious incident that occurred from using the device to the manufacturer (www.karex.com.my/contact-us) and the relevant national competent authority

Contraindications

There is no known effect of this lubricant on pregnant and/or nursing woman, woman with vaginal trauma/surgery and/or vaginal infection. Consult doctor before using this product.

Definition of Symbols (*All symbols used on product label conform to the harmonized standards, except below with their description*)

	Indicates that the product shall be used within 6 months after opening
	Indicates instruction to dispose empty packaging into rubbish bin

Innolatex (Thailand) Limited
E1-6, Export Processing Zone,
Southern Industrial Estate,
Village 4, Chalung Sub-district,
Hatyai District,
Songkhla, 90110,
Thailand

CE
0123