

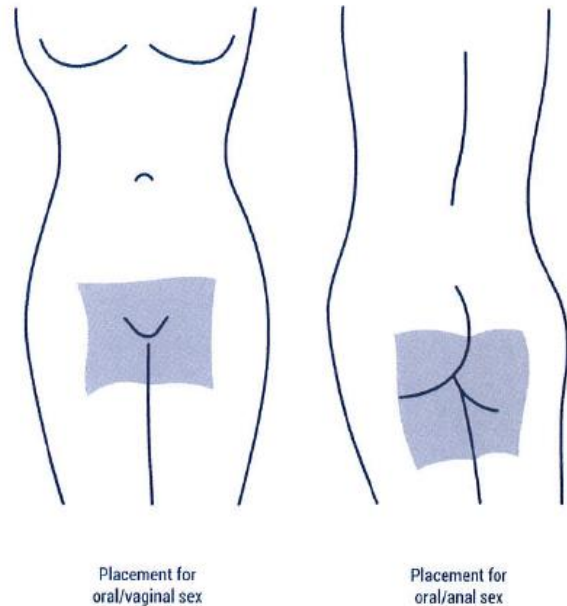
Overleaf/Inner Surface

Prophylactic dam is a medical device to be used by lay persons – Adult male or female.



[Brand] dams are thin, latex sheet designed as a barrier for use while performing oral/vaginal sex or oral/anal sex. When properly used, [Brand] dams may help to reduce the risk of transmission of many STIs, including syphilis, gonorrhoea, chlamydia, genital herpes and HIV. However, they cannot eliminate risk. During intimate contact, lesions and various body fluids can transmit STIs. Therefore, a [Brand] dam should be applied each time prior to sexual contact.

Instructions for Use

- Tear open the package carefully. Do not use fingernails, teeth or anything that can damage the dam. Remove the dam from the package.
- For oral/vaginal use, lay the [Brand] dam flat on the entire vulva, making sure to cover the vaginal opening and the clitoris. For oral/anal use, use a [Brand] dam to cover the entire anus. Hold the edges of the dam during use to ensure it stays in place.
- Optional: Moisten the vulva or anal area with water-based lubricant before putting the dam in place. Be sure to remove the lubricant from our fingers as this may make it difficult to hold the dam in place.
- Always use a new dam if you switch from oral/anal to oral/vaginal sex to reduce the risk of infection from harmful anal germs.



Warnings

- Do not use during penetrating penile/vaginal or penile/anal intercourse.
- If a lubricant is desired, use water-based lubricants, **DO NOT USE OIL-BASED LUBRICANTS**, such as those made with petroleum jelly, baby oil, body lotions, massage oils, butter, margarine, as these can damage the dam. So can some topical medicines that are applied to the penis or vagina. If you are not sure about the effect of the medicines you're using on dam, ask your doctor or pharmacist.
- If the rubber material is sticky, brittle or obviously damaged, do not use the dam.
- Do not stretch the dam.
- Store in a cool, dry place. Do not expose to direct sunlight **or symbols**   . Heat and light accelerate the degradation of latex.
- Dispose after single use. Do not share or reuse dams.
- This product contains natural rubber latex which may cause allergic reactions. If irritation or discomfort occurs, discontinue use and consult a doctor.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.











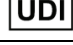

CE mark

Revision Date : 24th January 2022 (v.2)

Master reference information : K-IFU-07 (v.2)



Explanation on Symbols Used

| | | | | | |
|---|-------------------------------|---|--------------------------|--|------------------------------------|
|  | Store in a cool, dry place |  | Keep away from sunlight |  | Use by Date |
|  | Contains natural rubber latex |  | Manufacturer |  | Consult Instruction for Use |
|  | Do not reuse |  | Batch Code |  | Importer |
|  | Medical device |  | Unique Device Identifier |  | European authorized representative |
| Remark: Add in definition of any other symbols used on the packaging | | | | | |